


## Fitting Instructions

Wash your hands thoroughly with soap and water, or use hand sanitizer, before handling the respirator.

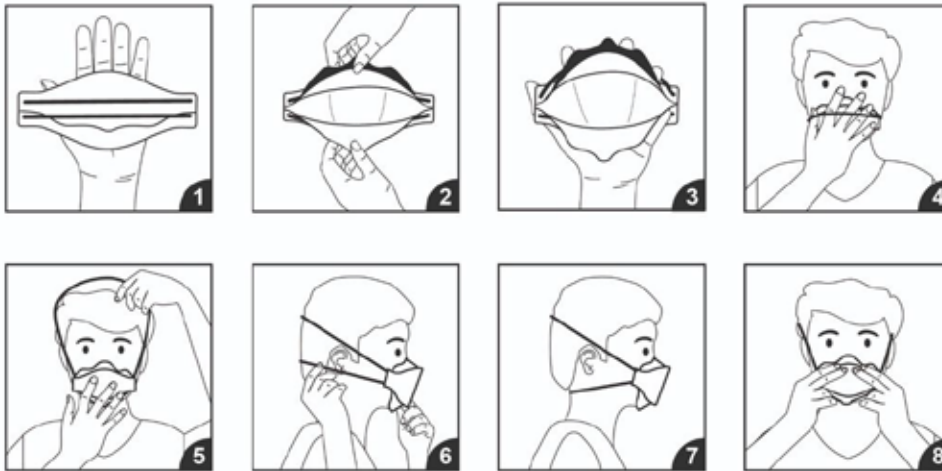

- 1) Remove the respirator from the packaging and hold with head straps facing upward.
- 2) Fully open top and bottom panels, bending nose foam around thumb at the centre. Head straps should be separate when panels are opened. Be sure bottom panel is completely opened and unfolded.
- 3) Place the respirator on palm of your hand and slightly tighten to make it a cup form while the top head strap positioned at the back side of your hand.
- 4) Place the formed respirator on your face.
- 5) Pull the top head strap back of your head while keeping the formed respirator on your face.
- 6) Pull the bottom head strap back of your neck while pulling the bottom panel under your chin.
- 7) The respirator should be placed as seen in the figure.
- 8) Place your fingertips from both hands at the top of the nose strip. With using two hands, mould the nose strip to the shape of your nose by pushing inward with your fingertips.

## Fit Check

Cover the front of the respirator with both hands being careful not to disturb the fit of the respirator.

Exhale and inhale sharply,

If air leaks around the nose, re-adjust the nose clip to eliminate the leakage,

If air leaks around the respirator edges, adjust panels and head straps.

If you cannot achieve a proper fit do not enter the hazardous area.